
 **WARNING:** Please mind potential fire risk and strictly follow the instruction manual for charging and storage. For disposal, please check your local authority's website for more information and dispose of the lithium-ion battery safely in accordance with the applicable legal requirements.

Smart VisualEar Pick

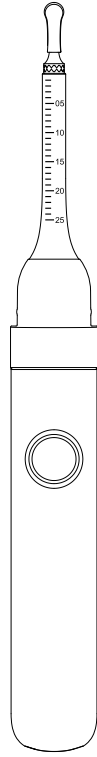User Manual
EN / DE / FR / ES / IT / JP

Catalog

English	01-03
Deutsch	04-06
Français	07-09
Español	10-12
Italiano	13-15
日本語	16-18

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
Download App

Step 1: Download the "Soulear" app
For iOS: Download and install the App "Soulear" from App Store.
For Android: Download and install the App "Soulear" from Google Play Store.
Optional: Scan the QR Code below to download the App.


Step 2: Connection Steps
 1. Press and hold the power button to turn on the device, and proceed to the next step after 5 seconds.
 2. Open the "Soulear" App, click the "Connect Device", follow the pop-up prompt and click to connect now to jump to the mobile WiFi interface.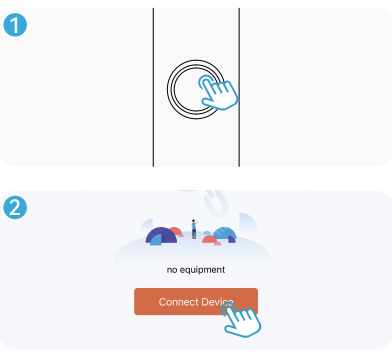


EN-01

3. Find "Soulear-XXXX" on the Wi-Fi settings interface and click to connect.
 4. After the phone displays "Soulear-XXXX" Wi-Fi is connected, return to the App again and click "Connected" to enter the device.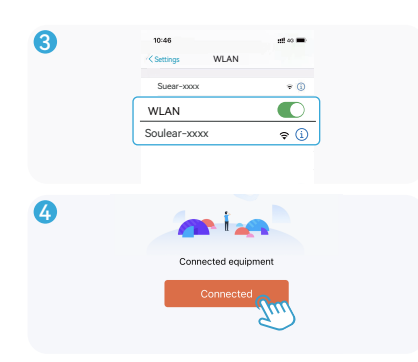


Accessories Installation


Earplug installation:
 Put the earplug on the corresponding scale position of the module tube to avoid inserting it too deep into the ear canal.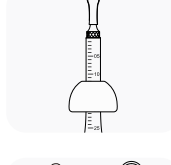

Ear spoon installation:
 Simply screw the metal ear spoon tightly and users can replace different ear spoon silicone covers according to different needs.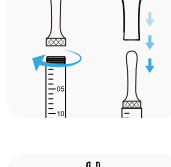

Tweezers installation:
 Install the metal or plastic tweezers kit into the body and secure it in place for normal use.

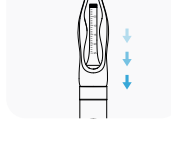EN-02

Troubleshooting

- Unable to store photos or videos to your phone**
Allow "Soulear" to allow permissions for photos, media, and files in your device.
- The camera does not respond and the camera does not light up**
Please charge your device.
- The app crashed**
Clear the background running on your phone and restart the app. Please charge your device.
- The camera red light flashes**
Please restart the device, open the APP, and connect to WIFI.
- WiFi connection is disconnected**
Please restart the device, open the APP, and connect to WIFI.
- When the phone prompts "Cannot join this WiFi, the network is temporarily unavailable, please try again later"**
When the above situation occurs, please try to turn off the product, turn it on again after 30 seconds, and reconnect to WiFi to use it.
- When connecting to WiFi, the Android phone prompts "Network access is full"**
When the above situation occurs, please try to turn off the product, turn it on again after 30 seconds, and reconnect to WiFi to use it.
- The ear spoon power indicator lights up red**
Please use the matching charging cable to charge.

*If the above instructions do not resolve the issue, please contact the seller's customer service via email and we will provide you with further solutions.

Product Parameters

Product Name	Smart visual ear pick
Input Current	5V=0.4A
Battery life	About 70 Min
Working environment	-10°C-45°C
Network standard	IEEE 802.11 b/g/n

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